Information for Use

Haemoglobin A1c Liquid Control Levels 1 and 2

European customers / Clients Européens / Europäische Kunden / Clientes europeos / Clienti europei / Europese klanten / Clientes europeos / Európai ügyfelek / Europeiska kunder / Avrupalı müşteriler / Европейски клиенти / Evropští zákazníci / Europæiske kunder / Europos klientai / Eurooppalaiset asiakkaat / Ευρωπαϊκοί πελάτες / 欧州のお客様 / Европейские клиенты / العملاء الأوروبيين

 **en** Instructions For Use in multiple languages are available on our website: www.canterburyscientific.com under the "Products and Instructions for Use" section. A paper version can be obtained by fax +64 3 3433342 or e-mail: info@canterburyscientific.com.

 This product contains human or animal source materials and should be treated as potentially infectious.

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Description and Intended Use

The extendSURE™ Haemoglobin A1c liquid controls **CONTROL** are prepared from human blood and contain normal haemoglobin, preservatives and stabilizers. The controls are formulated to give enhanced stability.

The Level 2 control is produced by a process of controlled *in vitro* glycation of non-diabetic blood.

The controls are designed for use as quality control materials to monitor the accuracy and precision of laboratory testing procedures for HbA1c quantitation for the following assay types: ion exchange HPLC, immunoassay and enzymatic procedures. For specific application in a particular assay refer to the analyser's operating manual. The control is not to be used as a calibrator.

Good laboratory practice dictates that a quality control program be established in all laboratories. This program consists of the routine assay of these controls by trained laboratory personnel and evaluation of the documented results against laboratory acceptable limits. If results are outside these limits re-run the control and investigate method parameters, environmental factors and techniques. If after re-running the control the result(s) continues to remain outside the acceptable limits, contact the manufacturer of the control for further control interpretation assistance.

Further good laboratory practice dictates that control samples and results be quarantined from patient samples and results.

Clinical Application of Haemoglobin A1c Measurement

The measurement of HbA1c is especially useful in the diabetic patient. Blood glucose levels fluctuate widely and instantaneous blood glucose testing does not reflect the averaged situation. The formation of HbA1c occurs slowly (about 0.05%/day) and continuously during the 120-day lifetime of the red cell. The measurement of HbA1c is useful to physicians as a long-term integral of blood glucose concentration and thus as a measure of the degree of control or self-management by the diabetic patient. However, each laboratory should determine a reference interval that corresponds to the characteristics of the population being tested. Each percentage point increase in HbA1c level corresponds to an increase in average blood glucose level of about 30 mg/dL or 1.7 mmol/L. As a general rule HbA1c levels above 10% represent poor diabetic control, whereas values between 6.5% and 7.5% are indicative of good control.

Kit Contents

REF B12397

Description:

- 12 vials of extendSURE™ Haemoglobin A1c liquid control Level 1 of 1.0 mL fill.
- 12 vials of extendSURE™ Haemoglobin A1c liquid control Level 2 of 1.0 mL fill.
- 1 Information Sheet.
- 3 Lot Specific Value Assignment Cards

Precautions/Warnings

WARNING

Potentially Biohazardous Material

Human red cells were used as a source material in the manufacture of the controls. Each unit was obtained from blood donors and tested by FDA accepted methods and found non reactive for Hepatitis B surface antigen, Anti-HCV, Anti-HIV 1 & 2 and Syphilis. No test method can offer complete assurance that the control containing human sourced materials will be absent of these and other infectious agents. Good laboratory practice dictates that all human source material should be considered potentially infectious and be handled with the same precautions used with patient specimens.

CAUTION

- Please **READ** through this **INFORMATION SHEET** before using the control materials.
- **FOR IN VITRO DIAGNOSTIC (IVD) USE.**
- **SAFETY CLOTHING** (glasses, gloves and laboratory coat) are **RECOMMENDED** when using these controls.
- **Discarded or spilt controls should be treated and disposed of as directed by your laboratory safety guidelines or by local ordinance.**
- **This PRODUCT CONTAINS a small amount of POTASSIUM CYANIDE. DO NOT INGEST.**
- **DO NOT USE CONTROLS BEYOND USE BY DATE and only use one control lot at a time.**
- **If there is evidence of MICROBIAL CONTAMINATION or BROWN DISCOLORATION or PRECIPITATION in the controls, discard the vial.**

Storage and Handling

The product must be stored protected from light, preferably in the original box.

Unopened

The extendSURE™ Haemoglobin A1c liquid unopened control vials should be stored in an upright position at 2-8°C. Controls can be used until the last day of the use by month shown on the bottle and is stable for 30 months from the date of manufacture.

Opened

The extendSURE™ Haemoglobin A1c liquid opened control vials should be stored in an upright position at 2-8°C. Once the controls are opened; they can be used for 30 days when stored tightly capped. The controls should not be frozen and thawed.

Control Procedure

The extendSURE™ Haemoglobin A1c liquid controls are ready for use and should be treated in the same manner as unknown patient specimens and run in accordance with the manufacturer's instructions accompanying the respective instrument/method. The controls exhibit column elution profiles comparable to patient whole blood haemolysates.

Value Assignment, Traceability Statement and Assay Values

The Control Mean [MEAN] and Range [RANGE] values for Haemoglobin A1c on the AU and DxC chemistry systems are detailed on the enclosed extendSURE™ Lot Specific Value Assignment Card [LOT SPECIFIC VALUE ASSIGNMENT CARD] in Units [UNITS] of % (NGSP Aligned [NGSP ALIGNED]) and mmol/mol (IFCC Aligned [IFCC ALIGNED]) as well as in units of g/dL and mmol/L. Also detailed to each card is a value for Total Haemoglobin in units of g/dL and mmol/L.

Values for Level 1 [Level 1] Control are printed on one side of each card with values for Level 2 [Level 2] Control printed on the other side.

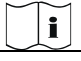
These assay values were derived from replicate analyses and are traceable and specific to the Manufacture's

Instrument/Haemoglobin A1c Reagent Assay

[MANUFACTURER'S INSTRUMENT/HAEMOGLOBIN A1c REAGENT ASSAY] using instruments and reagents lots available at the time of assay and are specific to this lot of Haemoglobin A1c liquid controls.

Since the assay values are dependent upon assay procedures as well as several other factors and because other assay systems may produce different values it is recommended that each laboratory should establish its own control limits from day-to-day use of the test. Any result that falls outside the limits established by your laboratory should be investigated.

In vitro Diagnostic Directive (IVDD, 98/79/EC and ISO 15223/1:2016) Symbols

				CONTROL	LOT
<p>Caution, consult accompanying documents Attention, consulter les documents joints Achtung, Begleitdokumente beachten Precaución, consulte la documentación adjunta Attenzione, consultare i documenti allegati Let op, raadpleeg de begeleidende documenten Cuidado: consultar os documentos anexos Vigyázat! Olvassa el a kíséző dokumentumokat. Försiktighet, konsultera medföljande dokument Dikkat, birlilke gelen dokümana bakın Внимание, консультируйтесь с сопроводительными документами Urozomení: Seznamte se s doprovodnými dokumenty Viktigt, se vedlagte dokumenter Demesio! Zr. pridruženi dokumentus Húmio, katos oheisiasiakirjat Προσοχή, συμβουλευθείτε τα συνοδευτικά έγγραφα 注意，参阅随附说明文档 警告、付属の文書を参照 Предостережение, сверьтесь с сопроводительными документами تنبیه، راجع المستندات المرفقة Urozomenie, pozrite sprievodnú dokumentáciu Thận trọng, tham khảo các tài liệu đi kèm</p>	<p>Biological Risks Risques biologiques Biogefährlich Riesgos biológicos Rischi biologici Biologische Risiken Riscos biológicos Biológiai veszély! Biologiska risker Bijoloji Riskler Биологични рискове Biologická rizika Biologisk fare Bioginijai pavojai Biologiset vaarat Βιολογικοί κίνδυνοι 生物危害风险 生物学的リスク Биологическая опасность المخاطر البيولوجية Biologické riziká Rủi ro Sinh học</p>	<p>Temperature limitation Température limite Zulässiger Temperaturbereich Limitación de temperatura Limitazione della temperatura Temperatuurgrens Limites de temperatura Hőmérséklet-korlátozás Temperaturgräns Sıcaklık sınırı Ограничение на температура Teplotní rozmezí Temperaturbegrænsning Temperatūros ribos Lämpötilarajotus Λεπτομερής θερμοκρασιακή температурные ограничения حدود درجات الحرارة</p>	<p>Consult Instructions for Use Consulter la notice d'emploi Gebrauchsanweisung beachten Consulte las instrucciones de uso Consultare le istruzioni per l'uso Raadpleeg de gebruiksaanwijzing Consultar as Instruções de Utilização Olvassa el a Használati utasítást. Läs bruksanvisningen Kullanma Talimatlarını Bakın Консультируйтесь с инструкцией за употреба Viz Návod k použití Se brugsanvisningen Zr. naudojimo instrukcijas Katso käyttöohje Συμβουλευθείτε τις οδηγίες χρήσης 参阅使用说明书 使用説明書を参照 Сверьтесь с инструкцией по эксплуатации راجع إرشادات الاستخدام Pozrite pokyny na použitie Tham khảo Hướng dẫn Sử dụng</p>	<p>Control Contrôle Controlle Control Controllo Control Control Kontroll Kontroll Kontrol Контроль Контроль Kontrola Kontrol Kontrolé Kontrolli Έλεγχος 质控品 コントロール Контроль سائل التحكم Kontrola Chất kiểm chuẩn</p>	<p>Batch Code (Lot) Code de lot (Lot) Chargencode (Chargenbezeichnung) Código de lote (Lote) Codice batch (lotto) Batchcode (partij) Código de Lote (Lote) Számkód (szarzs) Batchkód (lot) Küme Kodu (Grup) Код на серия (партида) Kód dávky (šarže) Batchkode (lot) Partijos kodas (serija) Eräkoodi (erä) Κωδικός παρτίδας (Παρτίδα) 批号 (批次编号) ロットコード Код серии (партия) كود التشغيل (الوط) Kód šarže Mã lô sản xuất (Số lô)</p>
EC/REP		 USE BY		REF	IVD
<p>Authorized Representative in the European Community Représentant agréé pour la C.E.E. Autorisierte EG-Vertretung Representante autorizado en la UE Rappresentante autorizzato nella Comunità europea Geautoriseerd vertegenwoordiger voor de Europese Gemeenschap Representante Autorizado na Comunidade Europeia Meghatalmazott képviselő az Európai Közösségen Auktoriserad representant i EG Ανγρω Βιλιγ'ηδεικι Yetkilii Temsilci Упълномощен представител в Европейската общност Autorizovaný zástupce v Evropském společenství Représentant i EU Igalizotasis atstovas Europos Bendrijoje Valtuutettu edustaja Euroopan yhteisössä Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα 欧共体授权代表 欧州共同体の正式代表者 Уполномоченный представитель в Европейском сообществе المنوب المعتمد في المجموعة الأوروبية Oprávněný zástupca v Európskom spoločenstve Đại diện được ủy quyền trong Cộng đồng Châu Âu</p>	<p>Manufacture Fabricant Hersteller Fabricante Produttore Fabricant Fabricante Gyártó Tillverkare Üretici Производител Výrobce Fabrikant Gamintojas Valmistaja Κατασκευαστής Παραγωγός メーカー Производитель الجهة المصنعة Výrobca Nhà sản xuất</p>	<p>Use by YYYY-MM A utiliser avant AAAA-MM Verwendbar bis JJJJ-MM Utilizar antes de MM-YYYY Utilizzare entro AAAA-MM Te gebruiken vóór JJJJ-MM Utilizar até AAAA-MM. Felhasználható: ÉÉÉÉ-HH Används före AAAA-MM YYYY-AA ile Kullanım Да се използва до ГГГГ-MM Použitelné do RRRR-MM Udløbsdato ÁÁÁÁ-MM Tinka iki MMMM-mm Viimeinen käyttöpäivämäärä KK-VVVV Ημερομηνία λήξης MM-EEEE 失效日期 YYYY-MM-DD (年-月-日) 使用期限YYYY年MM月 Исползовать до YYYY-MM استخدمه قبل من من من Použite do RRRR-MM Sử dụng trước Tháng - Ngày</p>	<p>European Conformity Conformité Européenne Europäische Übereinstimmung Conformidad Europea Conformità europea Europese conformiteit Conformidade com as normas europeias Európai megfeleléség Europeisk överensstämmelse Avrupa Uygunluğu Европейско съответствие Evropská shoda EU-konformitet Europos atitiklis CE-merkintä Συμμόρφωση με τους ευρωπαϊκούς κανονισμούς 欧洲合格认证标志 欧洲での適合 Европейское соответствие مستوفى الشروط الأوروبية Európska zhoda Tuần thủ Quy định Châu Âu</p>	<p>Catalog number Numéro catalogue Bestellnummer Número de catálogo Catalogusnummer Numero di catalogo Número de catálogo Katalógusszám Katalognummer Katalog numarası Katalogen номер Katalogové číslo Bestillingsnummer Katalogo numeris Luettelonumero Αριθμός καταλόγου 产品目录编号 カタログ番号 Номер по каталогу رقم الكالوج Katalógové číslo Số danh mục</p>	<p>In Vitro Diagnostic Medical Device Dispositif médical de diagnostic in vitro Medizinisches In-vitro-Diagnostikum Dispositivo médico de diagnóstico in vitro Dispositivo medico per diagnostica in vitro Medisch hulpmiddel voor in-vitrodiagnostiek Dispositivo médico para diagnóstico in vitro In vitro diagnosztikai orvosi eszköz Medicinsk anordning för in vitro diagnostik Каталожне изделие за ин витро диагностика In vitro diagnostický zdravotnický prostředek Medicinsk udstyr til in vitro-diagnostik In vitro diagnostikos medicinos prietaisais In vitro -diagnostinen lääketieteellinen laite In vitro διαγνωστικό ιατροτεχνολογικό προϊόν 体外诊断医疗器械 体外診断用医療機器 Медицинское устройство для диагностики In Vitro جهاز طبي للتشخيص خارج الجسم Katalógové číslo Thiết bị y tế chẩn đoán trong ống nghiệm</p>

Beckman Coulter, AU, UniCel and DxC are trademarks of Beckman Coulter, Inc and are registered in the USPTO. (United States Patent & Trademark Office)

71 Whiteleigh Avenue, Christchurch 8011, NEW ZEALAND Phone +64 3 343 3345 FAX +64 3 343 3342

EC/REP Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands